accumed

Sprague Rappaport Type Stethoscope

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www.accumed.ch

Replacing the Diaphragm

- Remove the desired assemble by gripping the assembly firmly and turning counter clock-wise.
- To loosen the diaphragm Retaining ring, remove worn diaphragm and replace with new.
- Reassemble disk, tighten by turning clockwise until tight.
- Reassemble diaphragm Retaining ring back onto chestpiece drum by turning clockwise until tight.

Cautionary Notes

- Please avoid extreme heat, cold, solvent, or oil.
- If sterilization is necessary, gas sterilize only.
- Do not use this device for diagnosis or treatment of any health problem or disease.
- Do not leave exposed to direct sunlight for prolonged period of time.
- Do not immerse the Rossmax EB500 Sprague in any liquid or subject to steam sterilization.
- Binaurals and tubing can be wiped down with alcohol or mild soapy water.

Operating Instructions

- Make sure the eartips are securely fastened onto the binaural.
- Insert the binaural into your ears carefully.
- Tap your finger on one of the diaphragms which is in the open position gently. If you tap the diaphragm which is in the close position, please rotate the head of the stethoscope 180o until it clicks into position, then you can hear the tap strongly and clearly.
- To replace a diaphragm or change to a bell, grasp the retaining ring (with entire assembly) and turn it counter-clockwise to unscrew. Then attach the diaphragm or the bell to the chest piece drum by turning it clockwise until secure.

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Warranty Card
This instrument is covered by a 2 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming date of purchase or the receipt. Accessories parts (Eartips, Diaphragms and Bells) are not included. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your local seller/dealer or www.accumed.ch.
Customer Name: _____
Address: _____
Telephone: _____
E-mail address: _____
Product Information
Date of purchase: _____
Store where purchased: _____

	Rossmax Swiss GmbH, <div>Widnauerstrasse 1, CH-9435</div> <div>Heerbrugg, Switzerland</div>		
			
			

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